

## Replacement Cushion/Elbow Kit (PB781-WE) Series for Zzz SG Series Full-Face CPAP Mask (PB7801/7801)

### Intended Use

The Zzz SG Full-Face Mask Replacement Cushion and Elbow Kits are intended to be used for replacement of worn or damaged cushions/elbows for adult patients who have been prescribed CPAP or bi-level therapy by a physician. It is used in conjunction with continuous positive airway pressure (CPAP) or bi-level in home, hospital, and institutional environments. **Rx Only.** Refer to your CPAP or bi-level device manual for details on settings and operational information.

### Contraindications

- This replacement kit may **NOT** be suitable for patients with the following conditions: impaired cardiac sphincter function, excessive reflux, impaired cough reflex, and hiatal hernia.
- It should **NOT** be used if the patient is unresponsive, obtunded, unable to remove the mask, or uncooperative.





### Warnings

This replacement kit is **NOT** suitable for providing life support ventilation.


- Keep all vent holes of the exhalation port clear.
- Ensure all packaging has been removed before using the replacement kit.
- Oxygen supports combustion. Oxygen **MUST NOT** be used while smoking or in the presence of an open flame. **DO NOT** smoke or light a flame in the presence of oxygen.
- Discontinue use of mask/replacement kit if any adverse reaction occurs. Consult your physician immediately.
- Contact your health care professional if chest discomfort, shortness of breath, stomach distention, belching, or severe headache are experienced.
- Avoid eating or drinking three hours before using the mask to minimize the risk of vomiting during sleep.
- **DO NOT** use bleach, chlorine, alcohol, or aromatic-based solutions - these may cause hardening and reduce the life of the product. They may also cause irritation to the skin.
- If any components (cushion, elbow) appear to be deteriorating at all (cracking, discoloration, tearing), discard and replace immediately.
- **USE ONLY** the cushion and elbow combination enclosed in this kit – **DO NOT** use other combinations of cushions and elbows. The two components are designed to work together **ONLY** – Failure to follow this instruction could result in insufficient exhaled CO2 porting.
- Refer to the User Manual that accompanied your MASK for warnings, instructions for use and proper assembly and fitting instructions. This User Manual applies **ONLY** to the replacement kit components (cushion and elbow).

### Cleaning Instructions

1. Detach headgear from mask. Handwash frame, cushion and elbow with mild soap and warm water. Rinse thoroughly and allow to air dry before reuse.
2. It is recommended that CPAP Mask Wipes are used daily to clean mask.

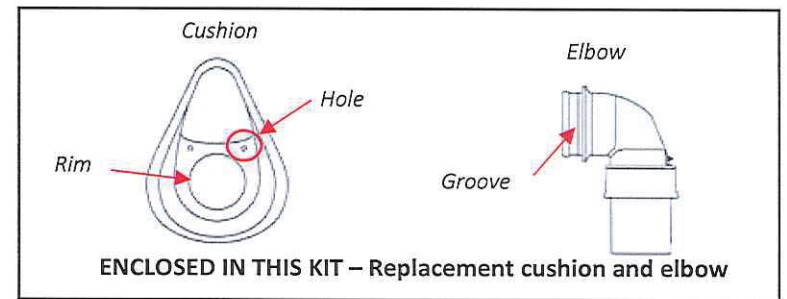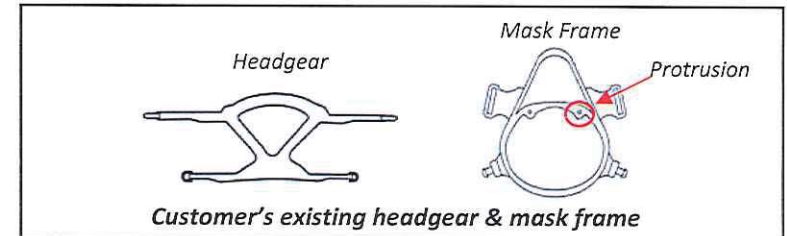

### Assembling & Disassembling the Mask

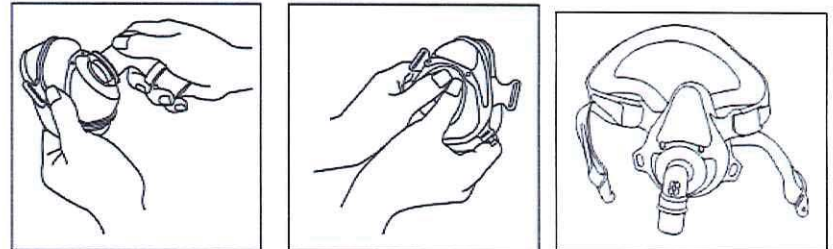

#### Assembly:

1. Insert the frame *protrusion* into the cushion *hole*. Repeat to attach both frame protrusions into cushion holes respectively.
2. Attach the *elbow* to the *cushion*. Make sure the silicon *rim* of the cushion's elbow hole fits into the *groove* of the elbow.

#### Disassembly:

1. Hold the *elbow* and pull its *groove* out from the *cushion*.
2. Push the *cushion hole* to detach from the *frame protrusion*.